Scan for more information

Check out our new learning platform Reliance U! [www.RelianceU.tech](http://www.RelianceU.tech)

#### RELIANCE

Reliance Orthodontic Products, Inc.  
1540 W. Thorndale Avenue,  
Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

EMERGO EUROPE  
Prinsessgracht 20, 2514 AP The Hague,  
The Netherlands

AUSTRALIAN SPONSOR  
EMERGO AUSTRALIA  
Level 20, Tower II, Darling Park, 201 Sussex St,  
Sydney, NSW 2000, Australia

ULTRA SEP™  
SINGLE BOTTLE  
SELF ETCHING  
PRIMER

INSTRUCTIONS FOR USE

CE 2797 V2 MAR.2022

RELIANCE®  
ORTHODONTIC  
PRODUCTS

RelianceOrthodontics.com

#### EN ENGLISH TRANSLATION

**INDICATIONS FOR USE:** UltraSEP is a self etching primer intended to be used for the preparation of an enamel surface prior to bonding with a light cure orthodontic adhesive.

**Performance Indicator:** Effective bond strength to enamel substrate for duration of treatment.

#### INSTRUCTIONS FOR USE:

1. Proph, rinse and dry the enamel.
2. Remove the cap from the UltraSEP bottle and dispense 1 drop into a mixing well. One drop will etch 6 teeth.
3. Saturate a microbrush and vigorously scrub the enamel surface for 5 seconds where the bracket will be placed and slightly beyond.
4. Dry the enamel surface for 2-3 seconds
5. Apply the bracket with light cure paste and light cure.

6. Replace the cap on the UltraSEP bottle.

⚠️⚠️ **PRECAUTIONS:** Flammable, Irritant

🌡️ **TEMPERATURE LIMITATION:**  
Store refrigerated. (2-8°C).

**WARANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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